

Looking after your Larval Therapy

BioBag daily care

- Do not immerse in water. Do not occlude
- Avoid sustained, direct pressure as this may occlude the larvae. Short periods for the purposes of mobilisation are permissible
- Daily change of the secondary dressings is recommended and when strikethrough is present
- Re-apply barrier where necessary to the peri-wound area (**fig.1**)
- Check larvae are viable at secondary dressing changes – movement of larvae and presence of dark red exudate indicate the larvae are alive (**fig.2**)
- Ensure damp gauze and an absorbent pad are replaced on top of the BioBag at each secondary dressing change (**fig.3**)
- Ensure that all outer/ dressings are not occlusive and are permeable to the air (**fig.4**)
- After 72 hours, reassess wound to decide on further treatment. If a further NEW larval treatment is required, schedule a new order
- If debridement is near completion and no further NEW larval treatment is required, plan follow on care/dressings as per specialist instructions or local formulary
- On removal, double bag and treat as clinical waste in line with your local Grade A Clinical Waste Disposal Protocol.

4 Day Treatment Cycle

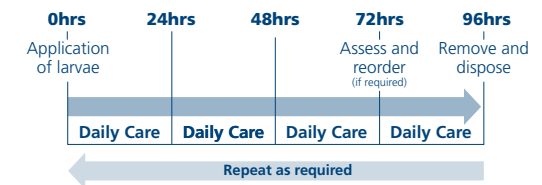


Fig.1 Re-apply barrier cream

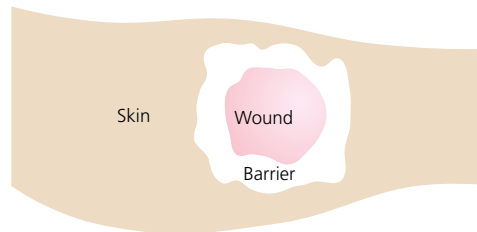


Fig.2 Check larvae are viable, reposition BioBag if necessary

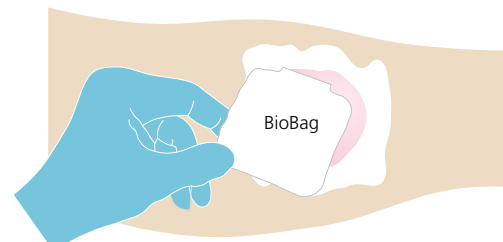


Fig.3 Place a moistened gauze swab over the BioBag


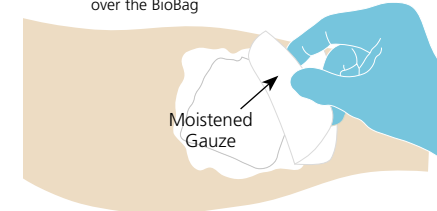


Fig.4 Secure the moistened swab and absorbent pad

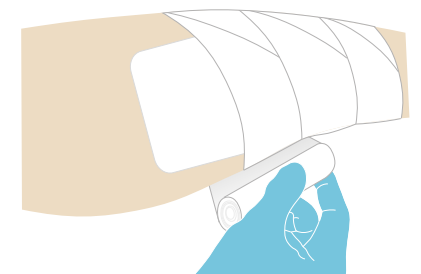

How to store and order

BioBag sizes and codes

	Code	Description
BioBag	BB50	2.5x4cm
	BB100	5x4cm
	BB200	5x6cm
	BB300	12x6cm
	BB400	10x10cm

Ordering Larvae

Orders received by us before 2pm will qualify for inclusive next day delivery, or a future planned date of your choosing.

Please allow time for your own internal procurement/pharmacy to process the order.

Telephone:

0845 230 1810

E-mail : orders@biomonde.com

Fax : 01656 668 047

Office Hours

Monday to Friday 8:30am – 5:00pm

For assistance outside working hours please call our **Clinical Helpline: 0845 230 6806**

Storage

- Keep in transit containers
- Store at a temperature of 6°C to 25°C (products do not need to be refrigerated)
- Must be applied by expiry date; usually the day after delivery. For optimal results apply on day of delivery.

www.biomonde.com